# medi Posture plus comfort

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Baumwolle, Elasthan, Polyamid, Polyethylen, Polyurethan

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# medi Posture plus comfort

## Intended purpose

Posture plus force is a T-shirt which corrects the wearer's posture. This product is to be used exclusively for treatment of the spine and is only intended for use on unbroken skin.

## Indications

- Poor posture, postural deviations or spinal deviations.
- Posture-related pain in the spine
- Muscle weakness and/or imbalance in the back
- Osteoporosis (in thoracic vertebrae and/or lumbar vertebrae)
- Hunched back with chronic back pain
- Secondary instability of shoulder girdle as a result of muscular deficits
- Asymmetrical scapula
- Deviations caused by scoliosis

## Contraindications

None known at present.

If applied correctly, there are no contraindications.

You should consult the doctor treating you before you use it if the following circumstances apply to you: Disease or injury to the skin in the area that the brace is used on, particularly when signs of infection (excessive heat, swelling or redness) are apparent.

## Mode of action / How to wear the shirt

The support that is woven into the fabric ensure that the trunk is kept upright, thus correcting posture. The trunk and shoulder muscles are activated and this works against deviations, muscle weakness and/or imbalances. This strengthening of the muscles is not noticeable but it improves mobility and

reduces pain.

If you experience excessive pain or your condition worsens whilst wearing the brace, stop wearing the product immediately and contact your doctor or the specialist supply store.

## Applying the shirt

Before you put the shirt on for the first time, ensure that you remove the protective film from the anti-slip stripes on both shoulders. These are on the inside of the shirt. Then put it on, as you would put on a normal T-shirt.

## Care instructions

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

## Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

## Material composition

Cotton, elastane, polyamide, polyethylene, polyurethane

## Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

## Disposal

The product can be disposed of in the domestic waste.

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