

RANGE OF POSTURAL AIDS FOR BED

RANGE OF POSTURAL AIDS MADE OF **MICRO-BEADS**

POSTURAL CUSHIONS MADE OF EXPANDED POLYSTYRENE MICRO-BEADS WITH SMALL GRANULOMETRY

MATERIALS



Expanded Polystyrene micro-beads with small granulometry



Removable Polymaille® integral cover

Foam maintenance :





Cleaning cover :



P9700B - UNIVERSAL «SMALL» & P9701B - UNIVERSAL «STANDARD» CUSHIONS

- ↳ This is the perfect supporting aid: it helps support the patient, relieves the pressure on the heels, protects the internal lateral knee areas, ankles and can be used as pillow.
- ↳ Used by itself or with other devices.
- ↳ Comes in two sizes, Small and Standard (see sizes on last page).

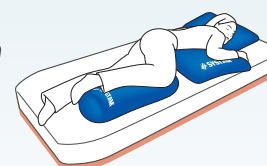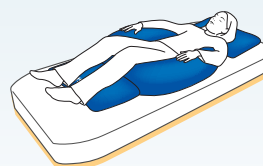





P9703B - ABDUCTION BLOCK FOR KNEES

- ↳ The simultaneous use of P9702 and P9703 cushions achieves a comfortable semi-Fowler position for the patient (back and pelvis at 30°) as well as a dorsal position by firmly holding the pelvis and abducted areas in place thanks to the abduction semi-circle cushion P9703.
- ↳ Reduces pressure on the internal malleolus and the inner sides of the knees.



P9702B - CYLINDRICAL CUSHION

- ↳ Can be used in various ways, such as placed under the knees to maintain lower limbs in a semi-Fowler position.
- ↳ When used with the abduction block (P9703), it firmly holds abducted limbs in position and protects the inner sides of the knees.

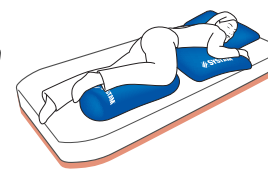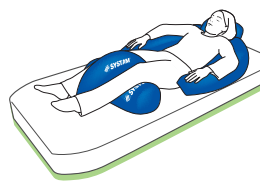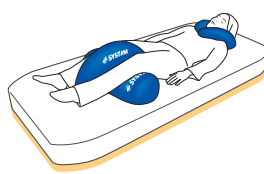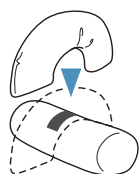

Item Code	Designation	Sizes cm (L x W)	Sizes inches (L x W)
P9700B1HW	P9700B - Universal cushion «small»	35 x 26	13,6 x 10
P9701B1HW	P9701B - Universal cushion «standard»	56 x 40	22 x 15
P9702B1HW	P9702B - Cylindrical cushion	Ø 20 x 60	Ø 7 x 23
P9703B1HW	P9703B - Abduction block for knees	62 x 32	24 x 12,4
P9704B1HW	P9704B - Ring cushion for neck	50 x 50	19 x 19
P9705B1HW	P9705B - Half-moon cushion	135 x 85	52,5 x 33
P9706B1HW	P9706B - Removable back cushion for half-moon cushion	58 x 40	22,5 x 15
P9707B1HW	P9707B - Lateral positioning cushion	180 x 54	70,8 x 21,6
P9708B1HW	P9708B - Half-moon cushion with back cushion	135 x 85 40 x 58	51,3 x 32,3 15,2 x 22
PKDEMOSAC01	Bag for transport		
PKDEMOPOSB01	Complete set with transport bag		

WARRANTY






RANGE OF POSTURAL AIDS MADE OF MICRO-BEADS

P9704B - RING CUSHION



- ↳ Comfortably supports the head both in sitting or semi-reclined position.
- ↳ Easy-to-use, self-gripping strip.

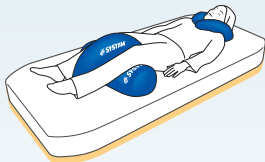

P9705B - HALF-MOON CUSHION


P9706B - ADDITIONAL BACK CUSHION FOR HALF-MOON CUSHION

- ↳ Holds the pelvis and upper body firmly in place when patient in semi-Fowler or dorsal position. Relieves pressure on elbows. Fitted with a strap for extra support P9706 enabling to maintain the head in position.
- ↳ Can also be used on a wheelchair.
- ↳ SYST'AM® exclusivity: equipped with the EasyShape® system that enables to maintain a position that is adapted to the patient's morphology.

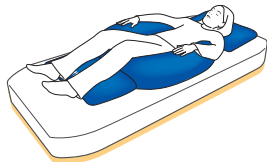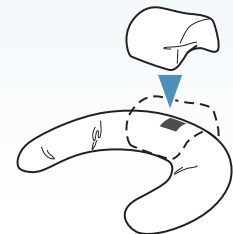

P9707B - LATERAL POSITIONING CUSHION

- ↳ Designed with integrated pillow for optimal comfort.
- ↳ Ensures comfortable positioning of the knees, pelvis and upper body when in lateral position.
- ↳ SYST'AM® exclusivity: equipped with the EasyShape® system that enables to maintain a position that is adapted to the patient's morphology.
- ↳ Available in one size, compatible with all patient sizes.

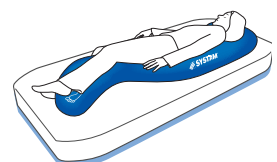

RANGE OF POSTURAL AIDS FOR BED

RANGE OF POSTURAL AIDS MADE OF **MICRO-BEADS**

PRODUCT «**ADVANTAGES**»

SYSTEM® MICRO-BEADS POSTURAL SUPPORTS FOR BED



EXPENDED POLYSTYRENE MICRO-BEADS

- ↳ Very light material: expanded polystyrene micro-beads with small granulometry for enhanced comfort and adaptability of support to the patient's morphology.
- ↳ Ignifugation treatment.



HOOK-AND-LOOP FASTENER

- ↳ Simple and efficient.

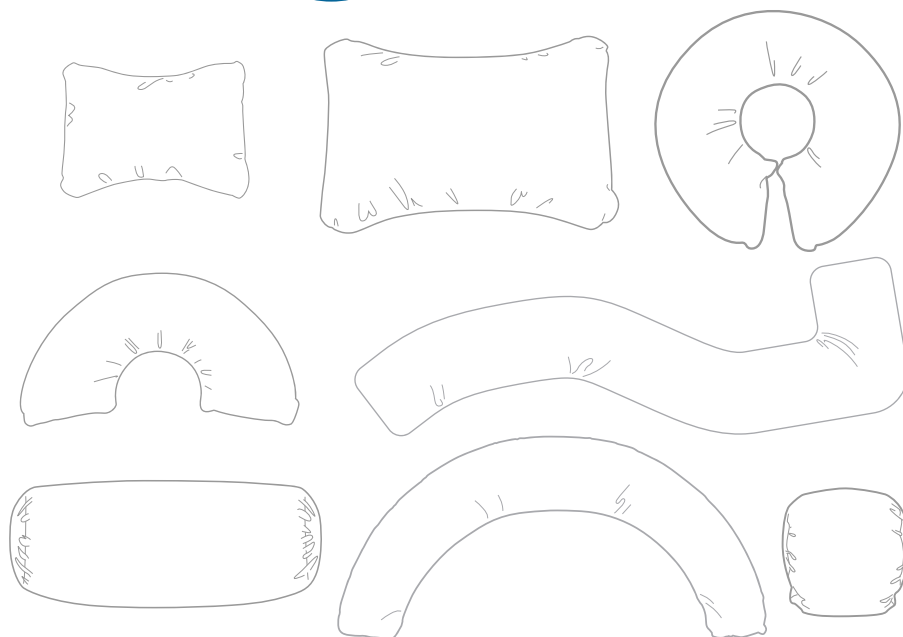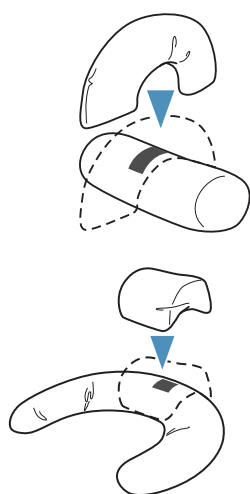

DELIVERED WITH TWO COVERS, ONE OF WHICH IS REMOVABLE

- ↳ Easy to wash
- ↳ Improved hygiene
- ↳ Extended durability of the cushion
- ↳ Adapted to multi-patient environments.

POLYMAILLE® COVER : BREATHABLE AND WATERPROOF

- ↳ Antiallergenic Polyurethane® coating on polyester jersey treated antibacterial, anti-germs, anti-fungal.
- ↳ POLYMAILLE® is extendable in two directions, which reduces shearing and friction effects.
- ↳ POLYMAILLE® offers high breathability and significantly reduces maceration..
- ↳ POLYMAILLE® is also totally waterproof for an increased control of hygiene.
- ↳ Washable at 95°, decontamination by means of cold-spraying.
- ↳ Fire retardant treatment.



CARRIER BAG

- ↳ with A4 document compartment.







RANGE OF POSTURAL AIDS MADE OF **MICRO-BEADS**

PRODUCT «**ADVANTAGES**»



SPECIFIC FEATURE FOR THE HALF-MOON CUSHION AND LATERAL POSITIONING CUSHION

EASYSHAPE®
S Y S T E M

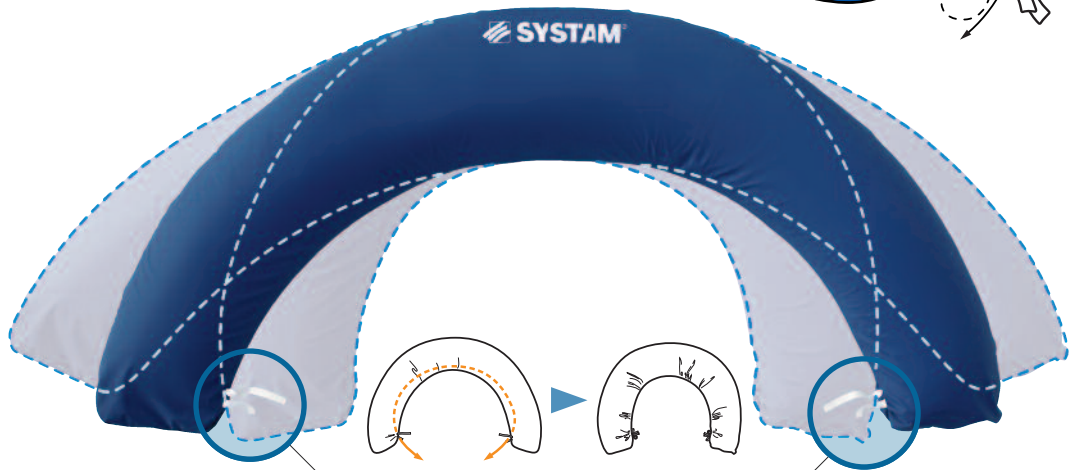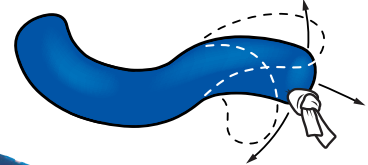

EASYSHAPE® :
EXCLUSIVE SYST'AM® ADJUSTMENT SYSTEM

- ↘ The straps attached to the half-moon cushion cover (P9705) and to the lateral positioning cushion cover (P9707) provide an easy way to adjust and to maintain the device in a position that is adapted to the patient's morphology, thereby increasing the patient's comfort and enhancing the ergonomics of the support.
- ↘ By pulling on the straps the curvature of the cushion increases and the shape is maintained by tying the straps.

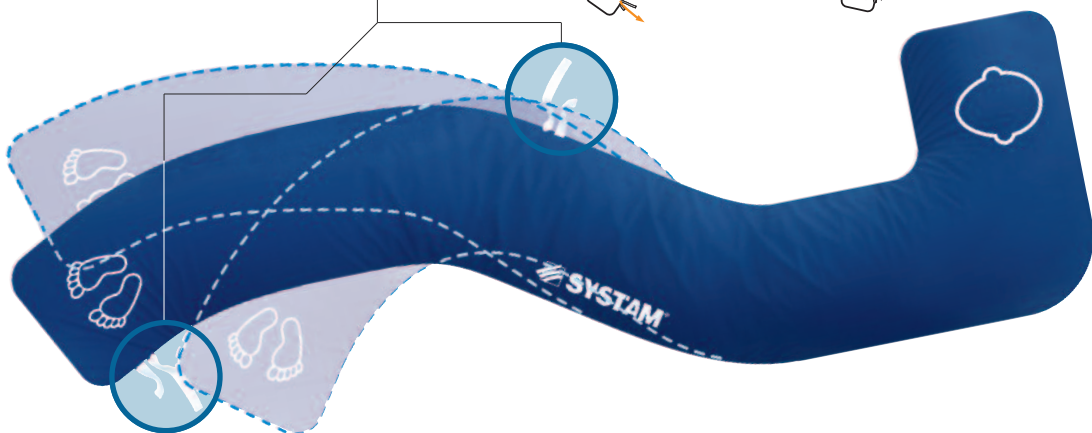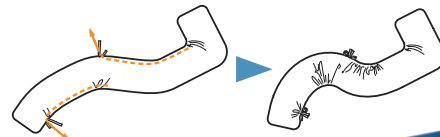

- ↘ Simple and easy to use, the EasyShape® system enables to adjust the device to better adapt it to the patient's morphology and to ensure that the position is maintained for an extended period of time.

