Otto Bock®

QUALITY FOR LIFE

CE

50S1

- (D)** Otto Bock® Dyna Ankle – dynamische Sprunggelenkorthese
nach Prof. Dr. Grifka
- (GB)** Otto Bock® Dyna Ankle – Dynamic Ankle Orthosis
designed by Prof. Dr. Grifka
- (F)** Otto Bock® Dyna Ankle – orthèse dynamique de la cheville
d'après Prof. Dr. Grifka
- (I)** Ortesi dinamica per caviglia – Otto Bock® Dyna Ankle
secondo Prof. Dr. Grifka
- (E)** Otto Bock® Dyna Ankle – órtesis dinámica tibioperoneotarsiana
según Prof. Dr. Grifka
- (P)** Otto Bock® Dyna Ankle – Ortótese dinâmica tíbio-tárcica
segundo o Prof. Dr. Grifka
- (NL)** Otto Bock® Dyna Ankle – dynamische enkelorthese
volgens Prof. Dr. Grifka
- (S)** Otto Bock® Dyna Ankle – dynamisk fotortos
enligt Prof. Dr. Grifka
- (H)** Otto Bock® Dyna Ankle – dinamikus bokaortézis
Prof. Dr. Grifka féle
- (CZ)** Otto Bock® Dyna Ankle – Dynamická ortéza hlezenného kloubu
dle Prof. Dr. Grifka
- (GR)** Δυναμική όρθωση αστραγάλου της Otto Bock®
σχεδιασμένη από τον Dr. Grifka

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1 Application

English

The dynamic ankle orthosis 50S1 Dyna Ankle is to be used **exclusively** for the orthotic treatment of sprains and ruptures of the anterior talofibular ligament.

2 Indications and Function

Sprains and ruptures of the deltoid ligament. Indication is given by the physician.

The dyna ankle orthosis places the foot in moderate pronation and dorsiflexion. The dynamic restraint limits plantar flexion and supination while preserving the range of motion in pronation and dorsiflexion. Positioning the orthosis with the dynamic strap provides relief of the anterior talofibular ligament.

The trimlines and straps are positioned to leave the outer ankle free, avoiding pressure points, thus allowing immediate application of the orthosis following injury. Heat or ice may be applied with the orthosis in place. It should serve its initial function as a post surgical immobilizer during the first two weeks. Then, subject to physician approval, the orthosis may be worn with a shoe during weight-bearing activities.

The Dyna Ankle is generally worn both day and night for stability for 6 weeks and is removed only for bathing.

The orthosis is waterproof and easy to clean and care for.

3 Size Selection of the Orthosis

The orthosis is selected according to the shoe size:

Article no.	Side	Shoe size	American Shoe Size	
			Men's	Women's
50S1=L35 – 37	left	35 – 37	4- 5	4- 6
50S1=R35 – 37	right	35 – 37	4- 5	4- 6
50S1=L37 – 39	left	37 – 39	5- 7	6- 8
50S1=R37 – 39	right	37 – 39	5- 7	6- 8
50S1=L39 – 41	left	39 – 41	7- 9	8-10
50S1=R39 – 41	right	39 – 41	7- 9	8-10
50S1=L41 – 44	left	41 – 44	9-12	10-13
50S1=R41 – 44	right	41 – 44	9-12	10-13

4 Construction

The dynamic ankle orthosis is ready for immediate trial fitting. An elastic strap runs medially from the lateral foot edge to the calf and across the top of the foot. The torsion cut at the distal end of the brace leaves the ball of the big toe free, thus lifting the lateral forefoot section.

5 Fitting the Orthosis

Apply the orthosis as follows

Place the foot in the plastic shell, sliding the heel to the back.

Pull the lower hook & loop strap from the inner side of the foot across the top of the foot through the outer plastic guide, then tighten the strap so that the heel will snugly fit to the shell.

Pull the upper hook & loop strap through the plastic loop and fasten it (fig. 1).

Fasten the hook & loop strap on the forefoot.

Slightly stretch the elastic functional strap and fasten it (fig. 2).

Adapting by the orthopedic technician/orthotist

The injection molded polypropylene sheet may be moderately modified (do not heat to temperatures exceeding 194 °F/90 °C). If it is necessary to modify the trimlines, the edges must be smoothed carefully since the material has minimal tear resistance when notched. The supplied straps are extra long and should be shortened to an appropriate length during fitting.

If pressure develops at the ankle or shin, the pads that are supplied with the orthosis may be glued to the strap (figs. 3 and 4).

6 Recommendations

To ensure the effectiveness of this orthosis, it must always be worn snugly.

In addition to proper application and regular check-ups with adjustments, patient cooperation plays an important role. The orthosis must be must worn as prescribed.

Since the orthosis is water-resistant, showers are possible while wearing it.

It is very important that the patient understand the important function of dynamic restraints.

A Patient Information sheet (646D36) is available for this purpose.

7 Warranty

The Otto Bock Warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturer's recommendations (see our general terms of sales, delivery and payment).

8 Declaration of Conformity

Otto Bock as manufacturer with sole responsibility declares that the 50S1 Dyna Ankle Orthosis conforms to the requirements of 93/42/EEC Guideline.

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Otto Bock HealthCare GmbH